

Defibtech Lifeline/ReviveR



Status Indicator: Status light



Status indicator when AED is ready for use:
Status light flashes green



Status indicator when AED is NOT ready for use:
Status light is red
No status light

Battery Life: 5 years
7 years
9V Battery Life: 1 year
Pad Life: 2 years

Shock Function:
Semi and fully automatic available
Pediatric Function:
Pedi pads available



Shock button



Shock indicator



On/off button

What to do if...

Status indicator is flashing red or solid red and/or is beeping periodically

♥ Press and hold the On/Off button for at least 5 seconds to initiate a manual self test.

Check pads and battery

- ♥ Ensure electrode pads are properly connected.
- ♥ Ensure the battery pack is inserted properly.
- ♥ Ensure the 9V battery is inserted in the main battery pack. 9V battery life is one year.
- ♥ Contact Defibtech technical support at 1-866-333-4241.

Status indicator is not showing any light

- ♥ Ensure the battery pack is inserted properly.
- ♥ Contact Via for a new battery and open a support ticket on onlineoversight.com.

AED is giving voice prompts

For all prompts related to battery and pads

- ♥ Ensure proper connection of pads and battery pack.
- ♥ Contact Via for a new battery and open a support ticket on onlineoversight.com.

For prompts regarding required service

- ♥ Contact Defibtech technical support at 1-866-333-4241.

If your AED is still not ready for use after following this guide, reach out to Via's service team at info@viaheartproject.org or 800-284-0125, or check the [full manufacturer manual](#) for more.