



HeartSine samaritan PAD 450P

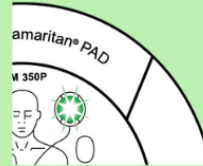

Status Indicator: Status light



Status indicator when AED is ready for use:
Status light flashes green

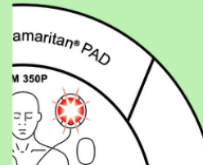

Status indicator when AED is NOT ready for use:
Status light flashes red
No status light

Battery Life: 4 years

Pad Life: 4 years

*battery and pad combo pack

Pediatric Function:

Pedi pads available

Shock Function:

Semi automatic



On/Off button



Shock button

What to do if...

Flashing red status indicator/ continual beeping, or no status indicator light

Check the expiration date on your Pad-Pak (on the underside of the AED)

If the expiration date has passed

♥ Contact Via for a replacement right away.

If the expiration date has not passed

♥ Press the On/Off button and listen for the voice prompt "Call for medical assistance"

♥ Then press the On/Off button again to turn off the device

♥ If this does not correct the problem, contact Stryker technical support at 800 327 0770

Voice prompts

"Warning low battery"

♥ Contact Via for a replacement right away and open a support ticket on onlineoversight.com

"Device service required"

♥ WARNING: If you hear this message during an emergency, seek an alternative AED.

♥ Contact Stryker technical support at 800-327-0770.

"Check pads"

♥ Ensure pads are properly connected. Remove and reinsert if necessary.

If your AED is still not ready for use after following this guide, reach out to Via's service team at info@viaheartproject.org or 800-284-0125, or check the [full manufacturer manual](#) for more.