



How to connect your ZOLL AED 3 to Wi-Fi

Step 1 - Turn on AED in Configuration Mode

- ♥ Press and hold the power button for 10 seconds to start in Configuration mode. Stop pressing when the ZOLL name displays.



Step 2 - Enter AED Management Mode

- ♥ Press the *Device Configuration* icon.



- ♥ Press the *Supervisor Access* icon.

- ♥ Enter the default password, "123456"



Step 3 - Set up Wi-Fi

- ♥ Press the *Wi-Fi Setup* icon.



- ♥ Select the option to edit the *Access Point List*.

Access Point List

Edit



- ♥ Press the *Wi-Fi symbol* next to SSID, then select your network and press *Ok*.



- ♥ Select the option to edit the *Password (Pre-Shared Key)*, enter the password for your Wi-Fi network, then press *Ok*.

- ♥ Press *Ok* on the *Access Point Settings* screen.

- ♥ Press *Save* on the *Access Point List* screen once your Wi-Fi network is listed.

Step 5 - Test your connection

- ♥ Select the option to edit *Device History Settings*.

- ♥ Select the option to edit *Selected Access Point* and set it to your Wi-Fi network.

- ♥ Press *Test Wi-Fi* to initiate a network connection test.

- ♥ If the test is successful, you can power off the device and return it to its service location.

- ♥ If the test is unsuccessful, double check that the password is entered correctly and that the right network has been selected.