

### ZOLL AED Plus



**Status Indicator:** Status symbol



**Status indicator when AED is ready for use:**  
Green check



**Status indicator when AED is NOT ready for use:**  
Red X



**On/Off button**



**Shock indicator/button**

**Battery Life:**

5 years from installation

**Pad Life:**

2 years (Stat Padz ii)  
5 years (CPR-D Padz)

**Pediatric Function:**

Pedi pads available

**Shock Function:**

Semi and fully automatic available

### What to do if...

#### Status indicator is a red X

##### Initiate a manual self test

- ♥ Press and hold the ON/OFF button for at least 5 seconds

##### Check pads and batteries

- ♥ Ensure electrode pads are properly connected.
- ♥ If the batteries were installed more than 4 years ago, contact Via for new batteries and open a support ticket on [onlineoversight.com](http://onlineoversight.com).
- ♥ If the batteries were installed less than 4 years ago, contact ZOLL technical support at 800-348-9011.

#### The AED is beeping once every minute

- ♥ The battery is low and needs replacement. Follow the steps above regarding batteries.

### How to open the battery compartment

#### Ensure the AED is turned off and place it face down to remove the battery cover

- ♥ Insert a small tool like a flat blade screw driver or a quarter into the two slots on the rear of the unit to depress the latches.
- ♥ Insert the tool into the groove on the bottom to lift the cover.

*If your AED is still not ready for use after following this guide, reach out to Via's service team at [info@viaheartproject.org](mailto:info@viaheartproject.org) or 800-284-0125, or check the [full manufacturer manual](#) for more.*