



ZOLL AED 3

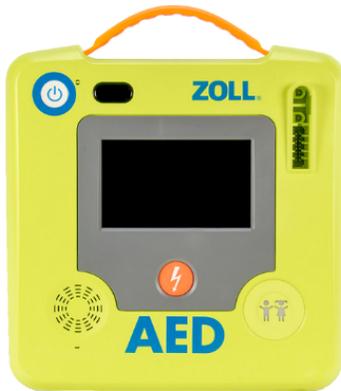

Status Indicator: Status symbol



Status indicator when AED is ready for use:
Green check

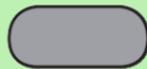

Status indicator when AED is NOT ready for use:
No symbol

Battery Life:
5 years

Pad Life:
5 years

Pediatric Function:
Pedi button

Shock Function:
Semi and fully automatic available



On/Off button



Child mode button



Shock indicator/button

What to do if...

Status indicator does not show green check

Initiate a manual self test

- ♥ Press and hold the ON/OFF button for at least 5 seconds

Check pads and batteries

- ♥ Ensure electrode pads are properly connected.
- ♥ Ensure the battery pack is inserted properly.
- ♥ Contact ZOLL technical support at 800-348-9011.

The AED is beeping once every minute

- ♥ The battery is low and needs replacement. Follow the steps above regarding batteries.

If your AED is still not ready for use after following this guide, reach out to Via's service team at info@viaheartproject.org or 800-284-0125, or check the [full manufacturer manual](#) for more.